

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\TA_VB17\MiriJuliPieri\T1\mp2rage_0.5iso

TA: 28:02 PAT: Off Voxel size: 0.5x0.5x0.5 mm Rel. SNR: 1.00 USER: mp2rage_wip602B

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	L2.4 A40.0 F20.0
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	15.4 %
Slices per slab	208
FoV read	224 mm
FoV phase	100.0 %
Slice thickness	0.50 mm
TR	5000 ms
TE	2.62 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	A24

Contrast

Magn. preparation	Non-sel. IR
TI 1	900 ms
TI 2	2750 ms
Flip angle 1	5 deg
Flip angle 2	3 deg
Fat suppr.	None
Water suppr.	None
2nd Inversion-Contrast	On
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	448
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	6/8
Slice partial Fourier	Off
Interpolation	Off
PAT mode	None
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off

B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved
Table position	H
Table position	0 mm
Inline Composing	Off

System

A24	On
V24	Off
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	--
Auto Coil Select	Off
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	220.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	L2.4 A40.0 F20.0
Orientation	Sagittal
Rotation	0.00 deg
F >> H	224 mm
A >> P	224 mm
R >> L	104 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Allowed
Contrasts	1
Bandwidth	250 Hz/Px
Flow comp.	No

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

Echo spacing	6.3 ms
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slab-sel.
RF spoiling	On
FFT Scale Factor	100 %
Line/Partition Swap	Off
Homodyne Phase Filter	Off
Flat Image	On
T1 Map	On
Division Image	Off
ExtInvPulseOn	On
OffResFreqInv	0
Invflipangle	2300